

The new **PLA-handles** in flat and high version will be reliably fixed by heat to **Erkoplast PLA**. Best already during the thermoforming process. No primer is needed.

(With LG-Primer and heat the PLA-handles can also be used for the previous types Erkoplast and Erkorit.)

The base of the PLA-handle and the ridge should roughly fit together. If necessary, soften and adapt the handle base by heating in boiling water, with the hot-air burner (177 540) or a hot air gun.

PLA-handles, 222 500, 50 pieces, high 

PLA-handles, 222 550, 50 pieces, flat 

PLA-handles



Thermoform and immediately place the PLA-handle onto the desired area and slightly press into the hot material. Finish the impression tray. Ready.

